

# ELECTRONIC CART

## Operating Instructions



### Disclaimers

Homak does not assume liability for any personal injury, arriving from the use of this product.

Homak warrants the product to be free from manufacturing defects for a period of 6 years from date of purchase.

One year warranty on all plastic parts. One year warranty on the electronic lock and locking mechanism.

US Patents pending, all rights reserved. Copyrighted 2001 by TriTeq LLC. Velcro is reg. US trademark.

Contact Lakeside's award winning Customer Loyalty Specialists  
for more information or to place your order today!

# 1-800-558-8565



4900 West Electric Avenue • West Milwaukee, WI 53219  
toll free: 800-558-8565 • phone: 414-902-6400 • fax: 414-902-6446 • www.eLakeside.com



## An Investment in Cart Security

## ***Please take a minute to read these instructions.***

HOMED Medical Products has taken on the commitment of bringing you the finest values in medical storage. This HOMED Electronic Cart is just one of the results of Homaks' efforts. It was developed to store your medical supplies securely with the flexibility of allowing you rapid access when you need it.

Homak also provides a full line of medical carts and cabinets at reasonable prices for your facility.

***Look us up on the web at [www.homakmfg.com](http://www.homakmfg.com).***

## **FEATURES of the HOMED Electronic Cart**

- All steel welded construction
- Five button Rapid access open
- Over one million key code combinations
- Variable length key codes 3 to 6
- Long life batteries – uses four “AA”
- External 9-volt power jumper – no need for expensive power supplies
- Durable Powder coating for a long-lasting finish
- Two motion opening sequence for added security
- Low battery detection light
- Quiet operation
- Easy to read, durable keypad
- 2-minute lockout feature to prevent tampering
- Detachable battery pack
- Steel reinforced electronics prevents tampering
- All electronics are housed inside the box
- Green Light button press indicator
- Easy to program code system
- Premium casters
- Built-in push handle
- Built-in stabilizing base
- Roller bearing glides
- Designed and manufactured in the United States

## **LOCKOUT**

If at any time you make an error, you can start over again by allowing the electronics to *Time Out* by leaving the **cart set for two minutes**.

DO NOT CLOSE THE CART after you enter your new User Code. Test the code by pressing it and confirming that the latch opened. (Look at the green light and/or listen to the latch opening.)

## **DEAD BATTERY PROTECTION**

### **Using the Battery Backup**

- 1)** The keypad has two small areas on the front, marked *Plus “+”* and *Minus “-”*. If the cart does not open or the green light does not light when a button is pressed, the batteries may be dead.
- 2)** Using a 9-volt battery, press it into the keypad at the two areas.
- 3)** Observe the polarity of the battery to the keypad.
- 4)** While holding the battery in place, press the *Factory Code* buttons and the cart will open.

### **Low Battery Indication**

While you are using the keypad to open the box, the battery condition is monitored. If they are close to being discharged, the red and green light will flash alternately, indicating *Low Battery*.


### **Error Indication / Time Out Feature**

To prevent unauthorized access and indicate an improper *Keystroke*, the electronics will flash red after the sixth *Keystroke*. Three unsuccessful attempts within two minutes will cause the unit to shut down for a two minute *Time Out*.

## STEP TWO

### Install Battery Pack

- 1) Open top drawer; attach battery pack to Velcro strip under the top of cart. DO NOT CLOSE CART.
- 2) Test the code again, pressing the five buttons per the factory code. You will hear the latch open. The green light will stay ON for three seconds while the latch is operating.
- 3) After you confirmed the latch operation you can now close the drawer by pushing it in flush with the front.
- 4) Close cart. To close cart, turn knob counter-clockwise to the closed position.

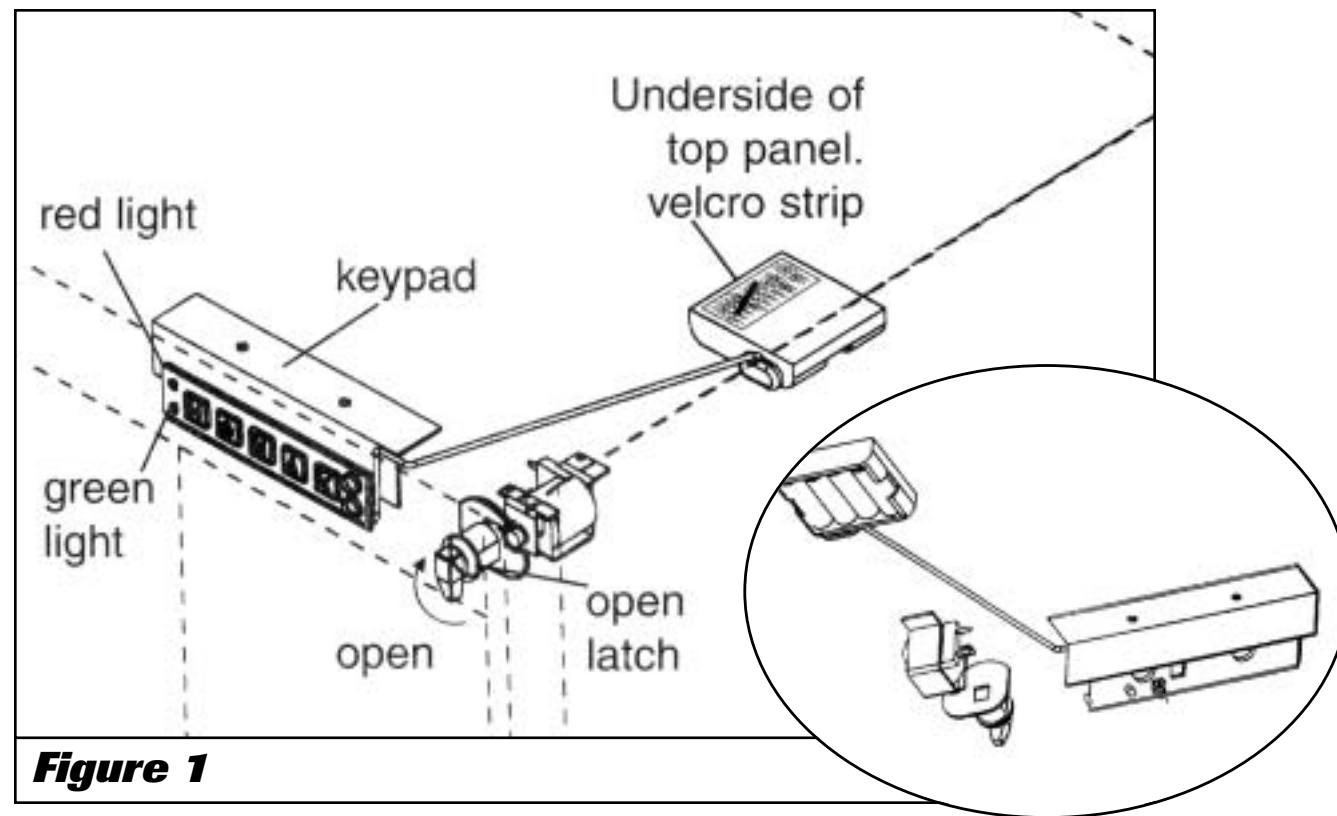

**Figure 1**

## STEP THREE

### Operation and Test

Press the factory code again. While the green light is on, turn knob clockwise. The cart is now open.

## STEP FOUR

### Programming

**Keystroke** A *Keystroke* can consist of pressing from one to three buttons at a time. ( i.e. a *Keystroke* can be a single button “1”, or two buttons at the same time “12”, or three buttons at the same time “123”. )

**Length of Code** You can select from three to six *Keystrokes* to activate your lock. ( i.e. you can program “1, 2, 3” or “1, 2, 3, 4, 5, 1” using individual buttons or by pressing “12, 12, 12” or “23, 23, 23, 23, 23”, using multiple buttons for each *Keystroke*. )

You may use any combination of *Keystrokes* and *Length of Codes* as desired. ( i.e. “12, 2, 12” or “45, 12, 3, 3” etc. within the definitions above. )


**Factory Code** The *Factory Code* is programmed in the electronics before it leaves Homak. It was supplied to you on a sticker included with these instructions. This code is erased when you enter in a new *User Code*. If the batteries fail, the lock will revert back to the *Factory Code*. The *Factory Code* is NOT the same for each box. If you lose or forget the *Factory Code*, please contact Homak. It is important that you keep these instructions or record the *Factory Code*.

## STEP FOUR

### Programming *continued*


**User Code** This is the variable length code you enter into your cart. It erases the original factory code. It is composed of three to six *Keystrokes*.

**Programming Button** This is a small click type button located inside the cart **behind the keypad underneath the letter "A"**. (See Figure 2.) To activate it, you press it towards the front of the box. You will need to HOLD the button in for four to six seconds to activate the programming function.

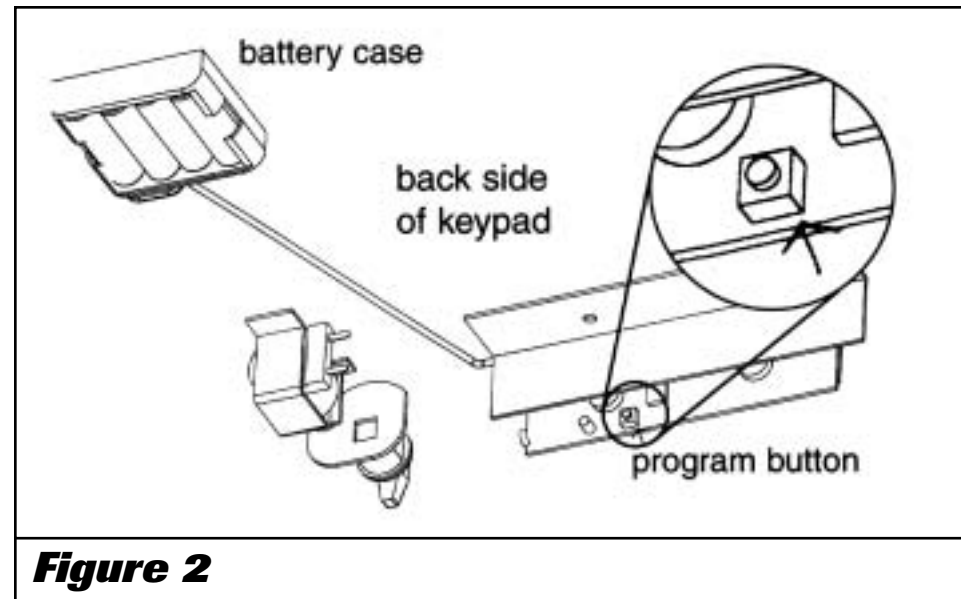

## Familiarize Yourself with the Cart

- Keypad
- Green Light
- Red Light
- Open Latch
- Programming button (Figure 2)
- Battery Case

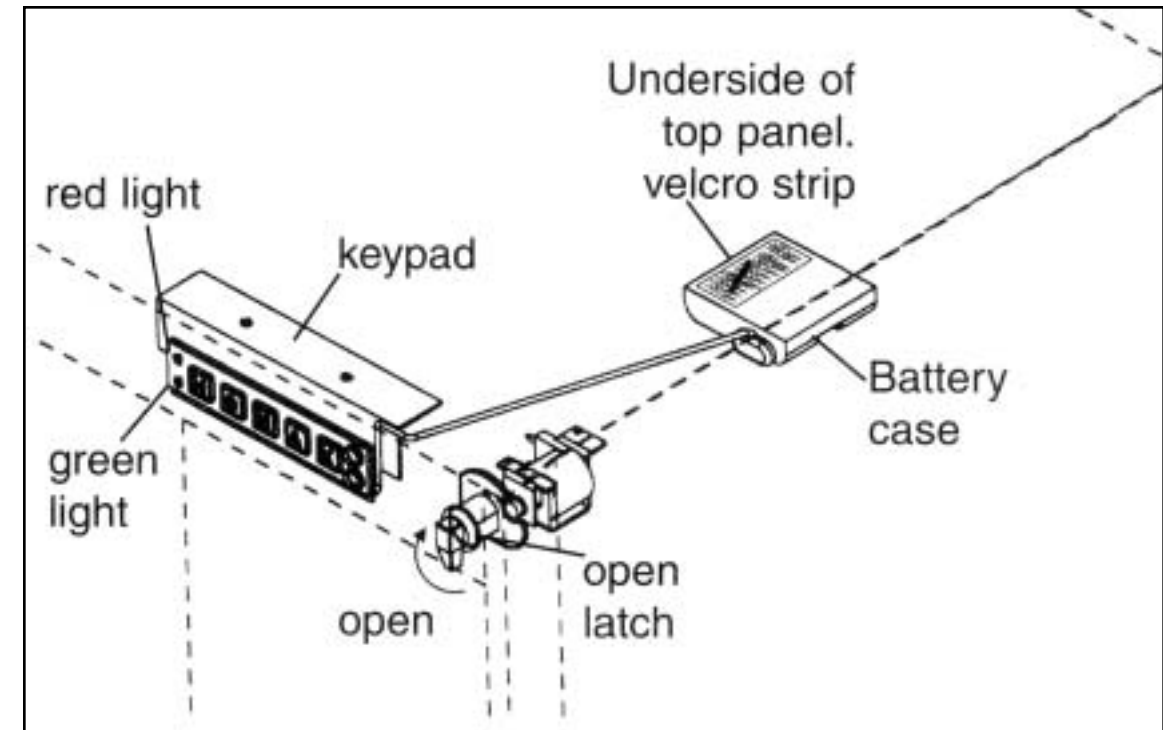

## STEP FIVE

### Entering a new User Code

- 1) Open cart using your *Factory Code*.
- 2) Locate the *Programming Button*, (See Figure 2.) press and hold the button for four to six seconds until you see the green light on the front panel stay ON. Remove your finger from the *Programming Button*.
- 3) Enter in your new *User Code*. After entering in your new *User Code*, you need to verify it.
- 4) Next press the *Programming Button* again until the green light flashes. Release the button after the light flashes.
- 5) Press in your new *User Code* again and press the *Programming Button* again. Hold the button in until you see the green and red lights flash alternately. This confirms you have entered in your new code successfully.

## STEP ONE

### Opening

- 1) After you have opened the carton, you will see the battery pack and installed batteries taped to the roof of the cart. **Enter the Factory Code** and open cart. (*Factory Code* found on the back cover page.)
- 2) When proper code has been pressed the green light will come on. Turn knob clockwise to the open position. You have 3 seconds to open the cart; after 3 seconds the solenoid will reengage, locking the cart.