



ASSEMBLY INSTRUCTIONS FOR
PREFERRED



MEDICAL CARTS
AND CABINETS

Contact Lakeside's award winning Customer Loyalty Specialists
for more information or to place your order today!

1-800-558-8565



4900 West Electric Avenue • West Milwaukee, WI 53219
toll free: 800-558-8565 • phone: 414-902-6400 • fax: 414-902-6446 • www.eLakeside.com


Thank you for investing in a Homed by Lakeside Preferred Medical cart. We at Lakeside have listened to your requirements in a medical cart and responded. The Homed Preferred Cart incorporates the design and features that you have requested. We are proud of our Medical Carts and are confident they will meet your expectations.

Before Assembly

Place a pad down in your work area before beginning assembly to prevent any scratching or denting the cart.

Unlock the cart and remove all hardware from the drawer. .


Exhibit 1

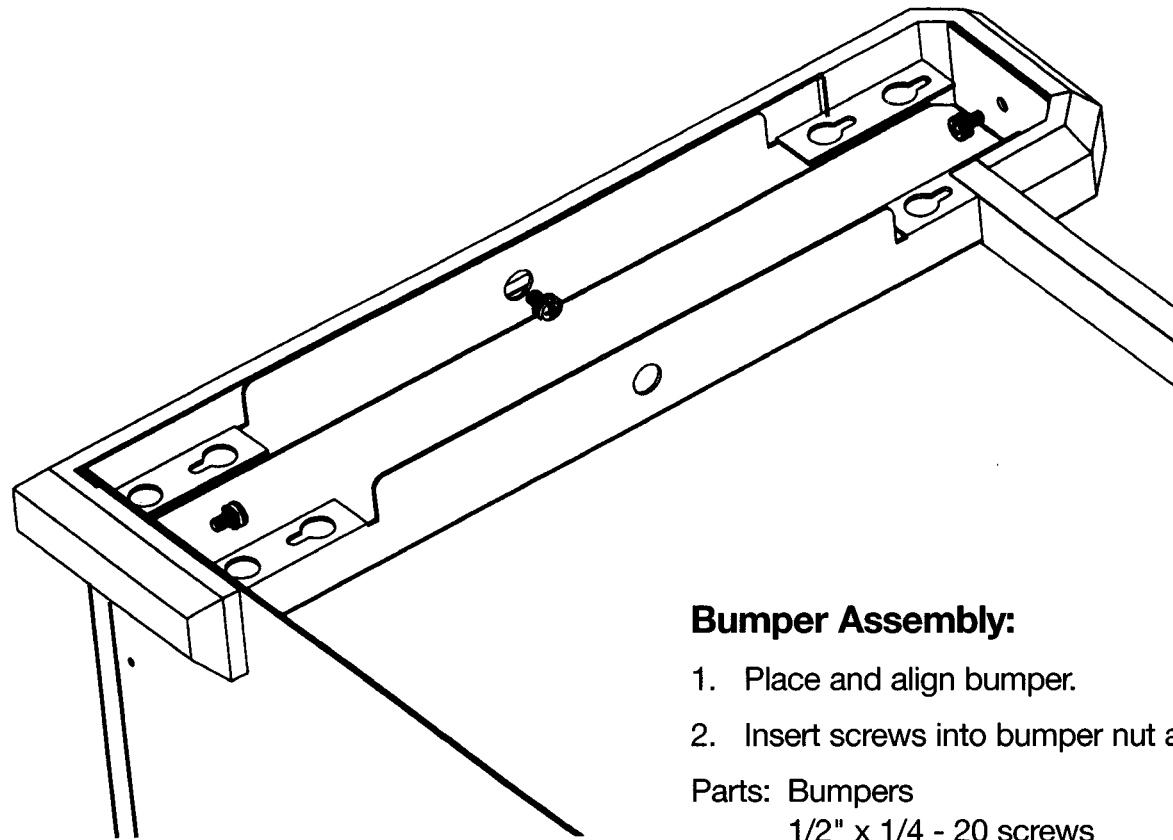

Bumper Assembly:

1. Place and align bumper.
2. Insert screws into bumper nut and tighten.

Parts: Bumpers 2 each
1/2" x 1/4 - 20 screws 6 each

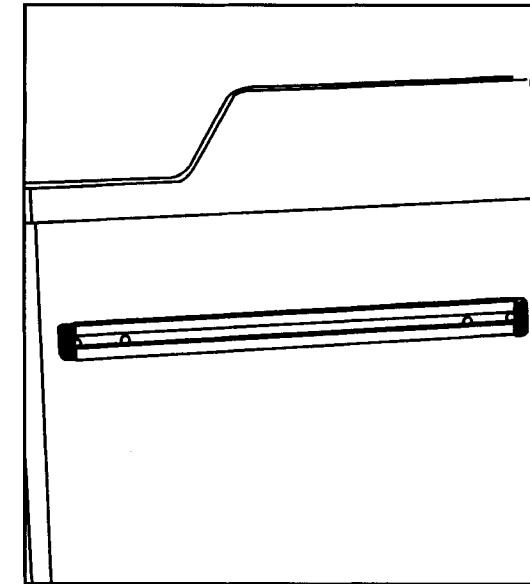


26039M ACCESSORY BAR

Parts: 10/24 x 1-1/4" Screws 2 each
Nylon Spacers 2 each
End Caps 2 each

Installation:

To accommodate your specific requirements, the 26039M accessory bar can be secured into two different locations on each side of the cart. See Exhibit 4 for details.

1. Align accessory bar with pre-drilled holes on side of cart.
2. Place nylon spacer between cart and bar.
3. Insert screw and tighten.
4. Insert end caps into bar.

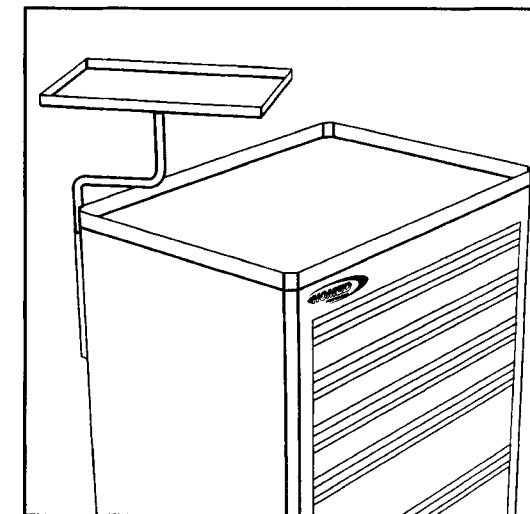

26033M DEFIBRILLATOR SHELF

Parts: #10-7/8" Phillips Head Screws 4 each

Installation:

Please note the Defibrillator shelf can be installed on either side of the cart.

1. Attach bracket to pre-drilled holes on cart. See Exhibit 3 for details.
2. Insert swing arm with washer-stop into bracket sleeve and tighten screws.
3. Insert other end of swing arm into sleeve on underside of shelf and tighten screw.

Maintenance


Your Homed by Lakeside Preferred Cart has been designed with simplicity and trouble-free service in mind. With a little care, this cart will provide years of service.

1. Clean your cart with a mild detergent or ammonia-based cleanser. This cleaning will prevent dirt from building up which may reduce the life and impair the operation of the cart. Wipe the ABS plastic top with a damp cloth and mild cleaning solution. Do not saturate the cart with any liquid. Clean up all spills with a dry, absorbent cloth.
2. Lubrication: Your cart's drawer glides need to be cleaned and lubricated once a year. This prevents dirt from accumulating which would prevent the drawers from working properly.

Exhibit 3


INSTALLING ACCESSORIES

All accessories will utilize a #10-7/8" long Phillips head screw to attach accessory to cart.

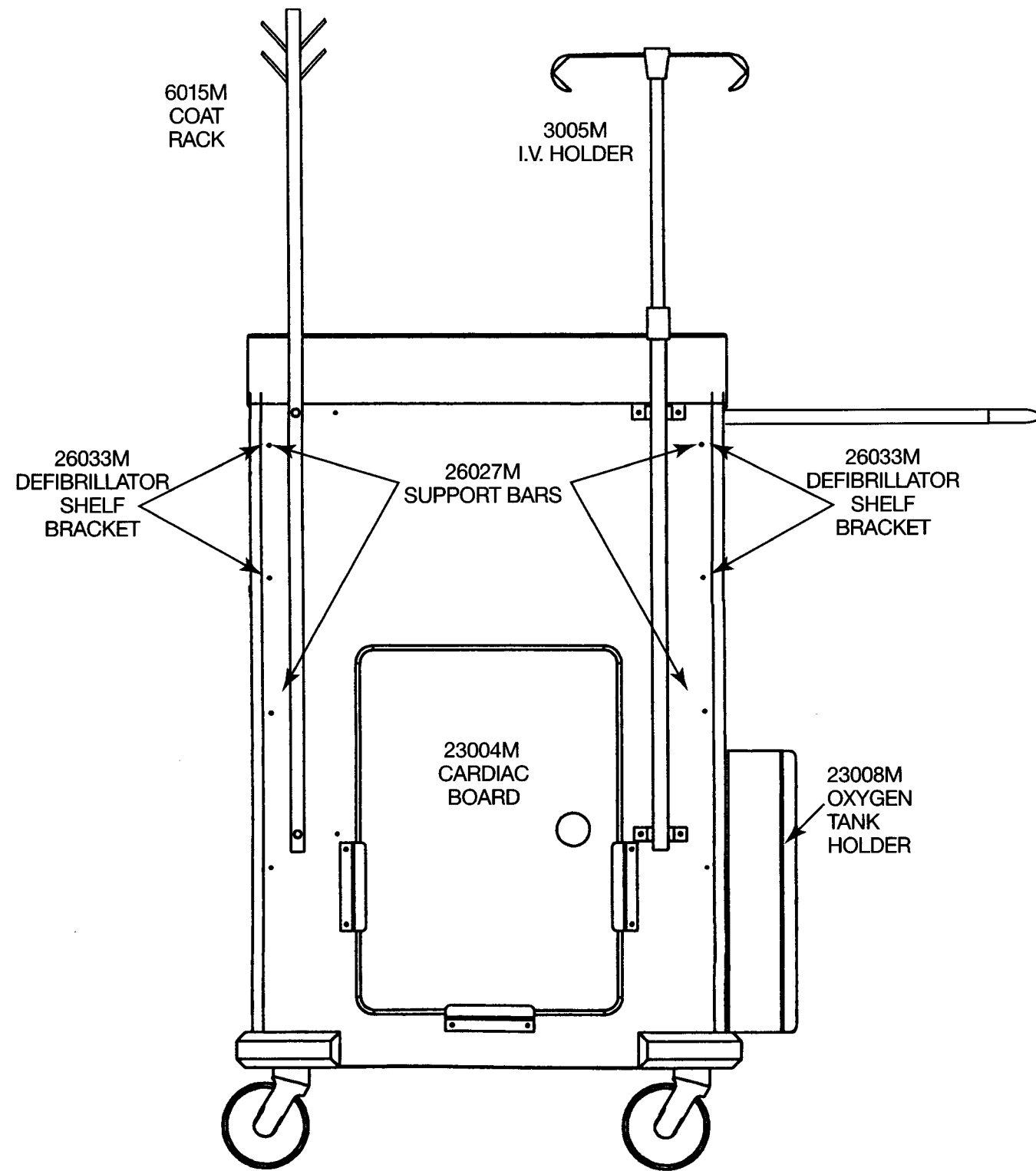


REAR VIEW

Exhibit 4

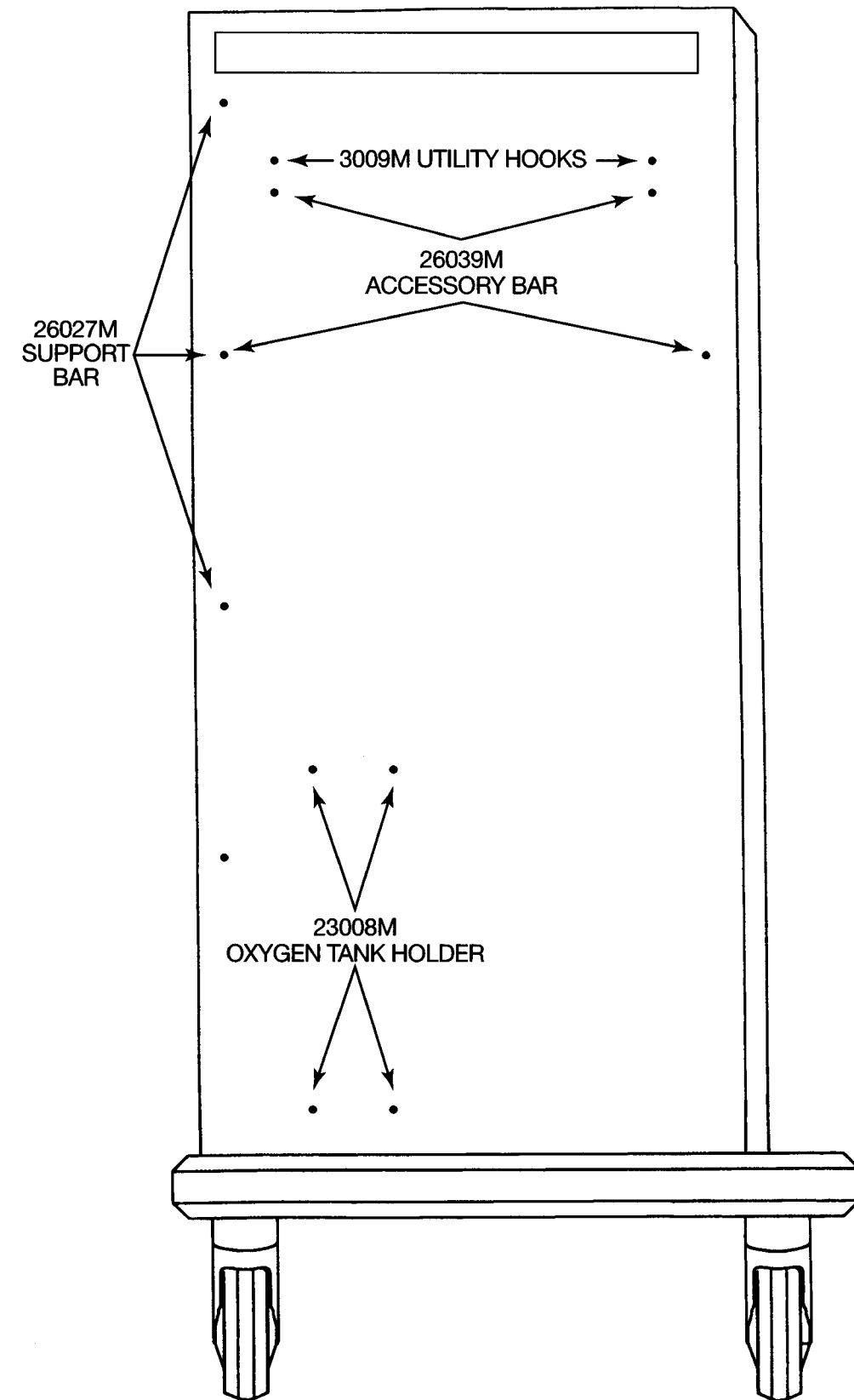

SIDE VIEW

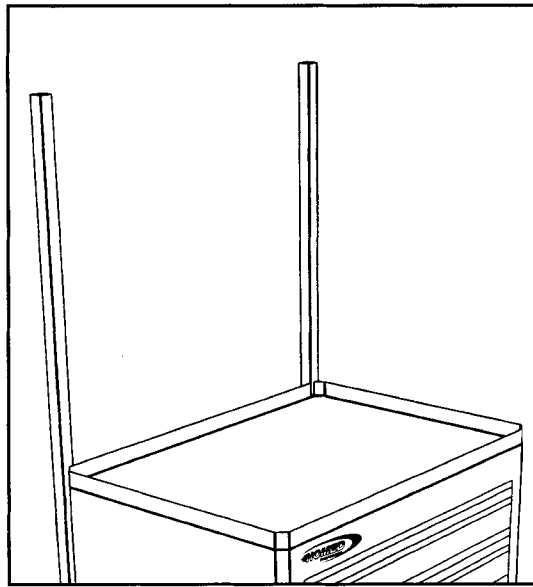


Installing Accessories

26027M SUPPORT BARS

Parts: #10-7/8" Phillips Head Screws 12 each
End Caps 2 each

Installation:

1. Align holes in angle iron on support bar with pre-drilled holes on cart. See Exhibit 3 for details.
2. Insert screw and tighten.
3. Repeat for remaining holes.
4. Insert end caps into support bars.

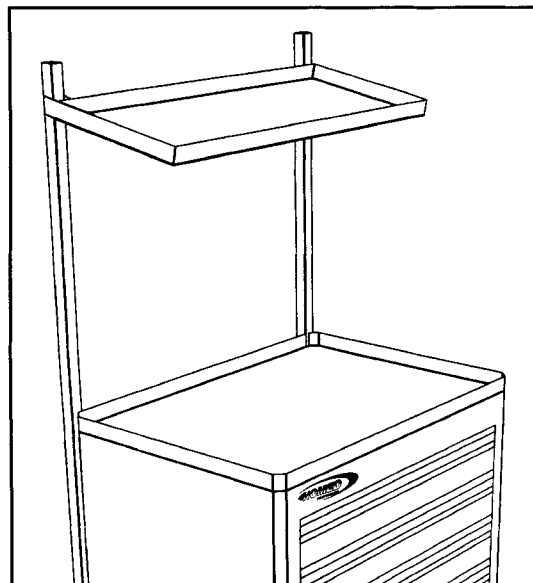


26020MS STRAIGHT SHELF, 26020MA ANGLED SHELF

Parts: Each Shelf contains:
1/4" - 20 x 1-3/8" Slotted Screws 2 each
Key Nuts 2 each

Installation:

1. Once the 26027M support bars are properly installed, set shelf onto the support bars by aligning the square tubing on shelf over the square tubing of the support bar.
2. Gently lower the shelf into position. Once you have the shelf where you want it, insert the screw with the threads facing the back of the cart.
3. Install key nuts and tighten.
4. Repeat for remaining holes and shelves.
5. Insert end caps into the tops of support bars.

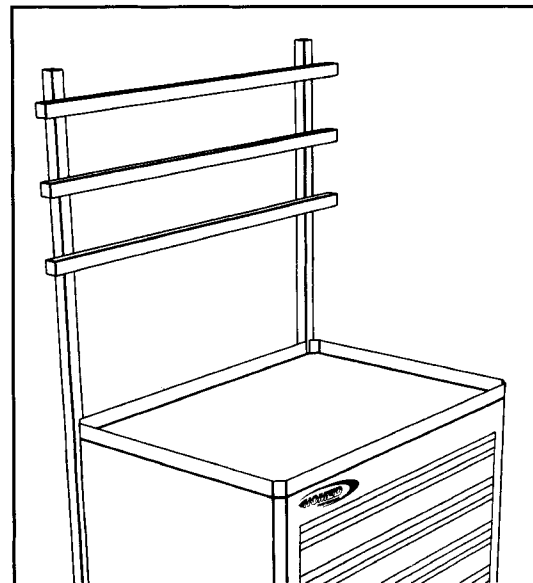

26021M TRELLIS BARS

Parts: 1/4" - 20 x 1-3/8" Slotted Screws 6 each
Key Nuts 6 each
End Caps 6 each

Installation:



1. Place the trellis bar in front of the 26027M support bar. Align hole in trellis bar with desired hole in support bar.
2. Place the 1/4-20 x 1-3/8" screw through the trellis bar into the support bar. Attach the key nut to screw. Nut should be on the backside of the support bar; tighten.
3. Repeat for remaining trellis bars.
4. Insert end caps into trellis bars and support bars.

Exhibit 2

Caster Assembly:

1. Place cart on flat side opposite push handle. Remember to place a pad down first to prevent any scratching or denting.
2. Position casters so that the 2 brake casters are in front position.
3. Assemble carriage bolt, washer and lock nut as shown. After the set of 4 bolts, washers and nuts are placed on casters, slide into position and tighten using 7/16" socket wrench.

Parts: 5/8" x 1/4 - 20 carriage bolts 16 each
Flat washer 16 each
1/4 - 20 lock nut 16 each

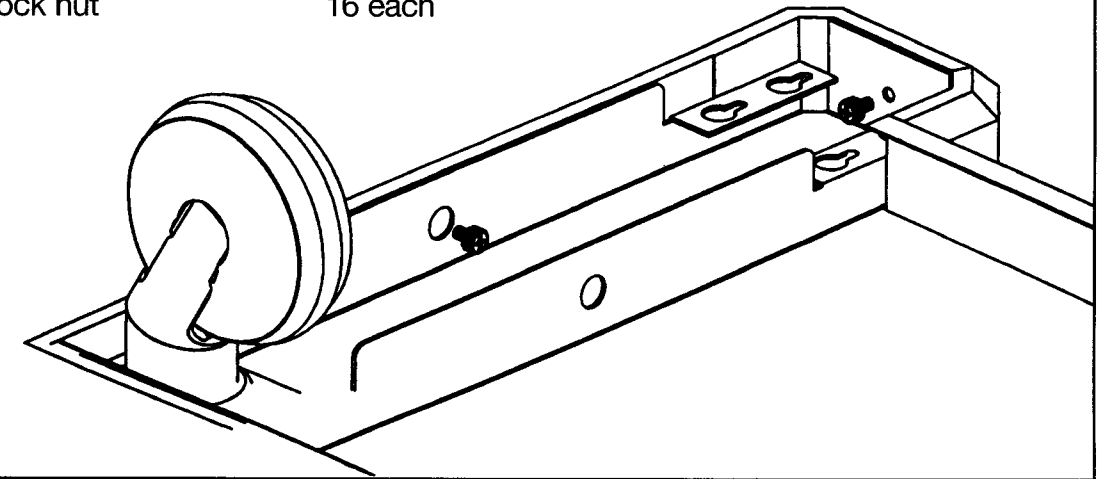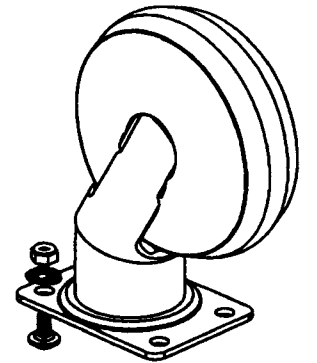


Removing Drawers

Slide drawer to full open position; push down on lever in right track and lift on lever in left track. While holding levers in this position, slide drawer off track.

Installing Drawers

Carefully line up glide on right and left sides. Insert drawer guide into glide. While holding glide in position, gently nudge into place. You may need help while inserting drawer. Make sure glides are held firmly in place while inserting drawer — failure to do so may result in drawer slipping out of place.

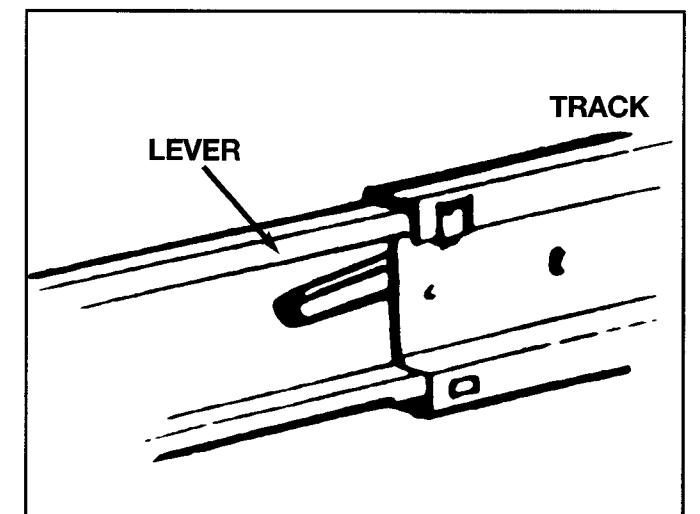

Locking Mechanism

Your Homed Preferred Cart is designed with an easy turn gang locking mechanism (non-emergency carts only). This locking mechanism is

designed so you can unlock the cart; open a drawer; then lock the cart — and as long as you keep one drawer in the open position, it will remain unlocked until all drawers are completely closed. The cart is then locked.